



easywrap[®]



 **INSTRUCTIONS ON REVERSE**



GENERAL INFORMATION

easywrap Liners are designed to be worn under the easywrap range of compression wraps, to protect the skin. The Liners are antimicrobial constructed with silver ions woven into them, this technology limits bacterial growth ensuring freshness, hygiene and comfort by helping the skin maintain its natural balance.

easywrap Liners are extremely soft and comfortable with excellent breathability and moisture management. Because of their high stretch they are very easy to apply and conform easily to most limb shapes.

STORAGE, GUARANTEE & DISPOSAL

easywrap Liners should be stored in a cool, dry, dark place without any exposure to direct sunlight or extremes of temperature.

Because the liners are worn and washed frequently they are subject to material ageing and natural wear. For Liners in use the period of guarantee is 6 months, however they should not be used beyond the expiration date printed on the outside of the package. At the end of their life garments can be disposed of as normal household waste.

PUTTING ON & TAKING OFF YOUR LINERS

CAUTION: Sharp fingernails, rings and bracelets can damage the garment. Loose threads or loops must never be pulled out or cut off.

Any open wounds should be appropriately covered. Ensure the limb is thoroughly dry before applying the garment.

Apply and remove your Liner as you would a pair of socks or stockings. Do not tug, force or over-stretch the garment. When applied run the palm of your hand up and down your limb to smooth out any creases. Any excess material can be folded over the top of the **easywrap** compression wrap.

WASHING INSTRUCTIONS

Liners should ideally be washed daily, or at least every other day.     

Alternating two or more garments improves the overall life of all garments.

To wash garments please use a normal mild detergent. **DO NOT USE LAUNDRY ADDITIVES**, such as fabric softener, optical brightener or stain remover. We recommend washing the garment in a 'laundry net'. Rinse the garment well. **DO NOT DRY ON A RADIATOR. DO NOT DRY THE GARMENT IN A DRYER.**

INDICATIONS

easywrap Liners are an accessory designed to be worn under **easywrap** garments to provide padding and skin protection.

Contains: 93% Polyamide, 7% elastane; with ionic silver inside yarn.

CONTRAINDICATIONS

Not to be used if you have an allergy or allergic type reaction to polyamide, elastane or ionic silver.

Haddenham Healthcare assumes no liability as a result of any contraindicated use of this product.